**miniLAND**  
BABY

# nasal care

Manual de Instrucciones • Instruction Manual • Manual de instruções  
Mode d'emploi • Bedienungsanleitung • Manuale d'istruzioni  
Instrukcja obsługi • Návod k obsluze • Руководство по эксплуатации

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## **1 INTRODUCTION**

Congratulations on the purchase of this ergonomic battery operated nasal aspirator. It will help you make them more comfortable; quickly and gently remove excess mucus.

Before using this device, please do read carefully all instructions in this manual. Please keep this manual as it contains important information.

## **2 CONTENTS**

- 1 Main Unit electronic nasal aspirator
- 2 Sizes of Silicone Tips
- 1 Cleaning Tubes
- 2 Spare Silicone Rings
- 1 Instruction Manual and guarantee card

## **3 SAFETY INSTRUCTIONS**

- 1. This device is not a toy; do not allow children to play with it.
- 2. This device is to clean stuffy noses; do not use it in baby's mouth or ears.
- 3. Always clean and dry thoroughly before each use and after each use.
- 4. This is not a waterproof device, please clean it referring to the section of "Clean Instructions".
- 5. Do not insert the silicone tip into the nostril too deep.
- 6. Do not use it on the baby for a long time. A few seconds will be enough.
- 7. Do not use this device if there is any wound, or symptom of inflammation or bleeding at the nose.
- 8. In the case of infection, fever, nasal irritation or hemorrhage, please stop using this device and consult your physician.
- 9. This device is restricted for personal use.
- 10. The silicone tip is the applied part.
- 11. No modification of this device is allowed. You are advised that changes and modifications not carried out on the equipment by the MINILAND technical service will not be covered by the product guarantee.
- 12. Duty cycle : 30 seconds ON / 2 minutes OFF
- 13. Replace batteries when the suction power becomes weak during use.


### **Battery Warnings:**

Use alkaline batteries only

- 1. Do not mix used and new batteries.
- 2. Do not mix alkaline, standard (carbon zinc), or rechargeable (nickel-cadmium) batteries.
- 3. Only batteries of the same or equivalent type as recommended should be used.
- 4. Batteries should be inserted with correct polarity.
- 5. Exhausted batteries should be removed from the device.
- 6. Be sure that all contact surfaces are clean before installing batteries.

7. Risk of explosion if battery is replaced by an incorrect type.
8. Remove batteries when stored for long periods of non-use.


#### 4 INSTRUCTIONS FOR USE

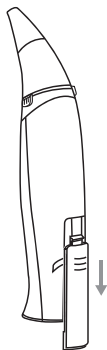1. Remove the battery cover as shown.


2. Insert 2 x 1.5V AA alkaline batteries (not included) as shown .
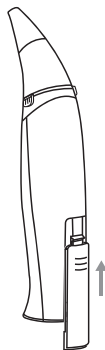


3. Place the battery cover as shown.

4. Use the suitable silicone tip based on the size of user's nostrils.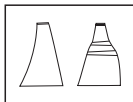

5. Be sure to place the silicone tip securely on the mucus cup.

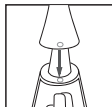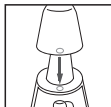
6. Gently insert the silicone tip into the nostril not too deep.


7. Press the button for a few seconds to collect mucus.

8. Repeat in the other nostril (if needed).

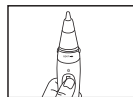
## Parts description
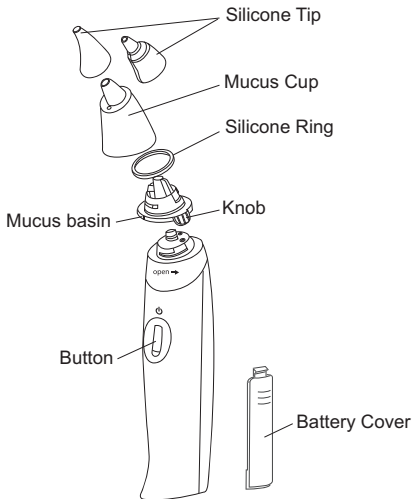
## 5 CLEAN INSTRUCTIONS

1. Remove the silicone tip.

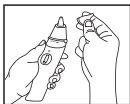
2. Use the finger and thumb to hold the 2 knobs of mucus basin and unscrew mucus basin from the main unit by turning it counterclockwise as "open ➡" on the collar and then pull up.


3. Hold the mucus cup upside down. Use the finger and thumb to hold the 2 knobs of mucus basin and turn it clockwise as "open ➡" on the mucus basin.




4. Remove the silicone ring from the mucus basin.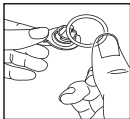


5. Wash all 4 parts in warm soapy water and then clean them with steam or boiling water.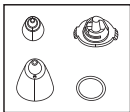

6. Assemble all the parts as shown below:

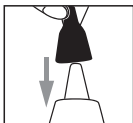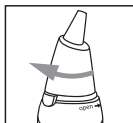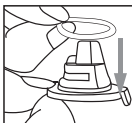
7. Be sure all the parts fit tightly together and line up the indication marks.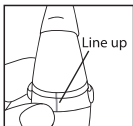


If you find mucus over the collar tap, please follow below instructions:

1. Please use attached cleaning tube. Put the cleaning tube to the suction hole.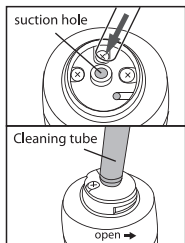

2. Put the cleaning tube in the water and press the button for 30 seconds.

3. Then lift the cleaning tube from the water and press the button again to pump out water inside till it is completely dry. Remove the cleaning tube and keep it for future use.

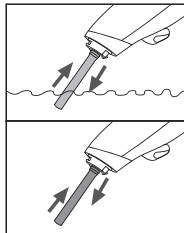## 6 TECHNICAL SPECIFICATION

- Operation environment:
  - Temperature: 10 °C~40°C
  - Humidity: ≤95%
- Storage & Transportation environment
  - Temperature: -10°C~50°C
  - Humidity: ≤95%
- Power: 2 x 1.5V AA alkaline batteries

## 7 BATTERY & PRODUCT DISPOSAL INFORMATION

- Remember, when you dispose of the nasal aspirator to remove the batteries and dispose of them responsibly.
- Do not mix products bearing the crossed out bin symbol with your general household waste.
- For the correct collection and treatment of these products take them to the collection points designated by your local authority. Alternatively contact the retailer who sold you the product.

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© Miniland, S.A. 2011
Miniland, S.A.
Miniland S.A. P.Ind. La Marjal I C/ La Patronal s/nº.